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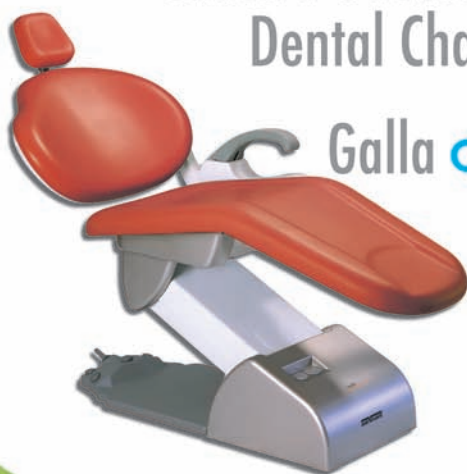
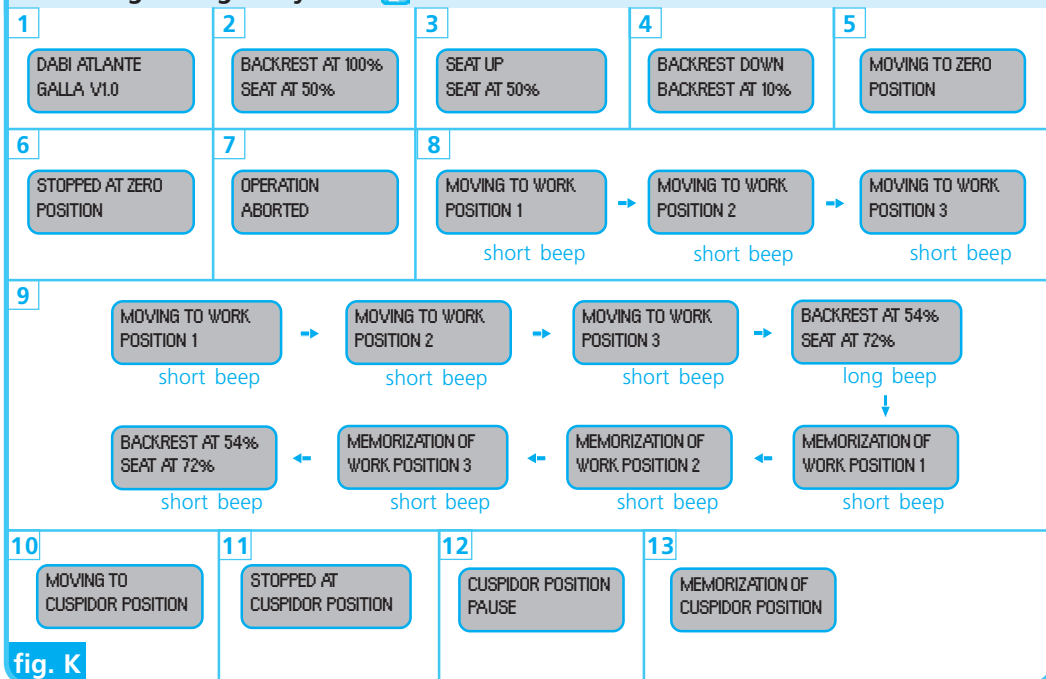
Messages - Digital System 

Contents

Presentation	12
Description	13
Galla Digital Symbol	13
Legend	13
Versa	13
Croma	13
Galla	13
Galla (digital option)	13
Foot Control	13
Installation	13
Procedures prior to equipment use/reutilization	13
Operation	13
Cleaning	15
Maintenance	16
Recommendations, care and warnings	16
Procedure in case equipment functioning is altered	16
Sensitivity to predictable environmental conditions in normal working situations	16
Procedures in case of equipment disposal	16
Transportation and storage conditions	16
After installation	16
Preparation prior to delivery	17
Dabi Atlante Authorized Service Network	17

Symbols	17
Final recommendations	17
Classification and type	17
Equipment warranty	17
Optional, serial and non-applicable items	18
Technical Features	18
Troubleshooting	19

Presentation

Dear Client,

Welcome!

We are really proud of your preference for our products. Your familiarization with the **Versa**, **Croma** or **Galla Chairs** will help maximize their performance and increase their durability and benefits. Therefore, before starting your equipment, be sure to read this manual very attentively.

Important

All the information, illustrations and specifications contained in this manual are based on data existing at the time of its publication. We reserve the right to make any modifications deemed necessary both on the product and in this manual without previous notice.

Description

The **Versa**, **Croma** and **Galla Chairs** have been designed to provide the dental surgeon with access to work regions in the operating field with all ergonomic conditions and maximum comfort for the patient.

Galla Digital Symbol

 Galla Digital Model

Legend

Versa (fig. A)

- ① Headrest
- ② Left arm
- ③ Seat
- ④ Backrest
- ⑤ Articulated right arm
- ⑥ Joystick
- ⑦ Foot control
- ⑧ Base
- ⑨ Motor protective cover
- ⑩ Master switch

Croma (fig. B)

- ① Headrest
- ② Left arm
- ③ Seat
- ④ Backrest
- ⑤ Articulated right arm (optional)
- ⑥ Joystick
- ⑦ Foot control
- ⑧ Base
- ⑨ Motor protective cover
- ⑩ Master switch

Galla (fig. C)

- ① 1- Headrest
- ② 2- Left arm
- ③ 3- Seat
- ④ 4- Backrest
- ⑤ 5- Articulated right arm (optional)
- ⑥ 6- Joystick
- ⑦ 7- Foot control
- ⑧ 8- Base
- ⑨ 9- Motor protective cover
- ⑩ 10- Master switch

Galla (digital option) (fig. D)

- ① Headrest
- ② Left arm
- ③ Seat
- ④ Backrest
- ⑤ Articulated right arm (optional)
- ⑥ Joystick
- ⑦ Foot control
- ⑧ Base
- ⑨ Motor protective cover
- ⑩ Master switch
- ⑪ Digital system (optional)
- ⑫ Class system (optional)

Foot Control (fig. F, G and H)

- ① Zero position
- ② Cuspidor activation
- ③ Spitting position
- ④ Light activation
- ⑤ Work position and setting
- ⑥ Joystick
 - ⑥a Backrest ascent
 - ⑥b Backrest descent
 - ⑥c Seat descent
 - ⑥d Seat ascent

Installation

This equipment must be unpacked and installed by an authorized Dabi Atlante technician; otherwise, warranty will be voided.

Procedures prior to equipment use/reutilization

Prior to equipment use/reutilization, follow the operation and cleaning instructions in this manual.

Operation

On/Off Switch

Set the master switch (item 10, fig. E) to "on" to use the **Chair** and to "off" after use.

 The master switch has an internal LED which remains lit/unlit when the **Chair** is on/off.

Chairs with a Spindle system:

In case a power failure occurs or the master switch (item 10, fig. E) is turned off while the **chair** is performing a movement (seat and/or

⚠ backrest), the zero-position key (item 1, fig. F and G) must be pressed when power supply is resumed or the master switch is turned on. Then, wait until the chair completes the movement in order to ensure proper functioning.

ℹ When turning the switch on, the display will indicate message 1 (fig. K) and after 6 seconds, it will show a message with the seat and backrest positions (message 2, fig. K).

Definition of the tone indication

The **Croma and Galla Chairs** (with a Spindle system) can be set so as to either emit a tone signal or not at each command. It is also possible to choose from either a standard tone signal or a polyphonic tone signal.

To set the tone indication:

1. Move the chair to the zero position by pressing the zero-position key (item 1, figs. F and G);
2. Keep the zero-position key pressed. A long tone signal will be emitted indicating the setting mode. Then, release the key after:

- The first beeping tone (single) is emitted for configuration without a tone signal;
- The second beeping tone (double) is emitted for configuration with a standard tone signal;
- The third tone signal (triple) is emitted for configuration with a polyphonic tone signal.

⚠ Independently of the configuration chosen, the work position commands will be followed by a tone signal and the movements made by means of the joystick will only emit a tone signal when the seat or backrest reaches the highest/lowest position.

Soft start system

The **Croma and Galla Chairs** (with a Spindle system) feature a soft start system which insures smoothness when both the start and stop movements of the chair seat and backrest are activated, thus providing better patient comfort and longer durability for the set.

Seat movement

Move the joystick (item 6, figs. A, B, C and D) vertically up and down (item 6c/6d, fig. H) to raise or lower the seat. To interrupt the movement, just stop moving the joystick.

ℹ The display will indicate the message referring to the movement that is being made as well as the current position (message 3, fig. K).

Backrest movement

Move the joystick (item 6, figs. A, B and C) sideways to the left/right (items 6a/6b, fig. H) to raise/lower the backrest. To interrupt the movement, just stop moving the joystick.

ℹ The display will show the message that corresponds to the movement that is being performed as well as the current position (message 4, fig. K).

Automatic movement to the zero position – ZP

Press the ZP key (item 1, fig. F and G) for the seat and the backrest to resume zero position simultaneously. This is the most comfortable position for the patient to get on or off the chair.

⚠ To interrupt the movement, move the joystick either way or softly press the ZP key.

For **Croma** and **Galla Chairs** featuring a spindle elevation system, the light will automatically go out when the zero-position is pressed.

ℹ When the zero position key is pressed, the display will show message 5, and when the movement is completed, message 6 will be shown. In case the movement is interrupted, the display will show message 7 (fig. K).

Light activation

Press and release the key to turn the light (item 4, fig. F and G) on or off.

⚠ The light reflector must be connected to the chair circuit.

To alternate the light luminous intensity on the **Croma** and **Galla Chairs** featuring a spindle elevation system, keep the key pressed until the light reaches the desired intensity.

Automatic movement to the work position – WP (chairs with a hydraulic elevation system)

Press the WP key (item 5, fig. F and G) for the seat and backrest to move to this position.

To interrupt the movement, move the joystick in either direction or press the WP key again.

ℹ In order to activate one of the memorized WPs, keep the WP key pressed until the display shows the message with the desired WP (message 8, fig. K), and then release the key. The movement will start.

When the movement is interrupted, the display will show message 7 (fig. K).

Activation of the work positions (chairs with a spindle elevation system)

Press and release the work-position key – PT (item 5, fig. F and G) after:

- The first beeping tone (single) to move the **Chair** to work position 1;
- The second beeping tone (double) to move the **Chair** to work position 2;
- The third beep tone (triple) to move the **Chair** to work position 3.

After the key is released, the chair seat and backrest will simultaneously move to the selected work position and the light will be set to the selected intensity.

To interrupt the movement, activate any other control.

Memorization of the work position (chair with a hydraulic elevation system)

Move the chair backrest and seat to the desired work position by using the joystick. Press the WP key continuously until a long beeping tone is emitted and then release the WP key. Repeat the operation to reset the work position.

Memorization of work positions (chairs featuring a spindle elevation system)

These **Chairs** have three programmable work positions. Programming is performed as follows:

1. Move the chair to the desired work position by using the joystick.
2. Keep the work-position key (WP) (item 5, fig. F and G) pressed. Three tone signals will be emitted followed by a long one, indicating the programming mode. Release the key after:

- The emission of the first beeping tone (single) for memorization of working position 1;
- The emission of the second beeping tone (double) for memorization of working position 2;
- The emission of the third beep tone (triple) for memorization of working position 3.

⚠ It is possible to memorize the light intensity simultaneously with the work position by setting it at the desired intensity before starting memorization.

Memorization of work positions (optional Digital Galla)

Position the backrest and the seat in the desired position using the joystick. Press the WP key continuously until the display shows the message that corresponds to the position that you want to be memorized (message 9, fig. K) and then release the WP key. Thus, the desired position will be programmed in the memorized position chosen.

Spitting position - SP (optional Digital Galla)

To move the backrest to the spitting position (SP), softly press the SP key (item 3, fig. G). The display will indicate message 10 (fig. K).

As soon as the movement is concluded, the display will show message 11 (fig. K). To resume the previous position, softly touch the SP key.

⚠ To interrupt the movement, a quick touch to the SP key while the movement is in progress is enough. The display will show message 12 (fig. K).

Memorization of the Spiting Position

Move the backrest to the position to be memorized and then press the SP key continuously until a long beeping tone is emitted followed by a short one. When message 13 (fig. K) shows on the display, release the key.

Automatic activation of the cuspidor water flow

The water flow to the cuspidor is automatically activated when the SP movement begins and remains active until the WP is resumed.

⚠ When the movement is interrupted, the flow of water to the cuspidor remains activated. If you want to interrupt water activation, use the cuspidor control or the key on the foot control (item 2, fig. G).

Activation of the cuspidor water flow using the foot control

Press the water activation key on the cuspidor (item 2, fig. G) with a quick touch. To interrupt the water flow, press the key again.

Arm movement

Versa Chair

The right/left arm (item 5, fig. A) of the **Versa Chair** assembled on the right/left-handed version is articulated. Move it up for the patient to enter/exit the chair and then down during the treatment.

Croma and Galla Chairs (optional)

The right/left arm (item 5, figs. B, C, and D) of the **Croma and Galla Chairs** assembled on the right/left-handed version features a lateral movement. Pull it out for the patient to enter/exit the chair and then move it back during the treatment.

Longitudinal movement of the headrest

Turn the knob (fig. I) located on the rear side of the backrest counterclockwise to unlock the backrest. Set it in the desired position and turn the knob clockwise to lock it again.

Positioning of the bi-articulated headrest

Turn the knob (fig. J) counterclockwise to unlock the mechanism. Place the headrest in the desired position and turn the knob clockwise to lock it again. Ensure that set has been properly tightened in order to prevent it from moving during use.

Utilization of the heel protector

The heel protector prevents damage to the upholstery. Its attachment is made by Velcro (adhesive nylon strips) on the lower side of the upholstery.

Cleaning

Clean the equipment surface using a clean soft cloth moistened in Aplic Odonto by Dabi Atlante or another product with similar chemical properties.

⚠ The use of other chemical products is not recommended as it may damage the equipment.

Maintenance

Preventive maintenance

In order to reduce equipment faults and increase durability, call Dabi Atlante Authorized Technical Assistance and set a regular preventive maintenance plan.

Corrective maintenance

In case the equipment presents any faults that are not listed in "Troubleshooting", contact Dabi Atlante Authorized Technical Assistance.

- ⚠ Do not open the equipment and/or try to repair it as such action may aggravate the problem or even generate further malfunctioning.

Circuit diagrams, list of parts, components, others

In case they are necessary, ask Dabi Atlante for electric diagrams and/or lists of parts, components or other information, which will be supplied by agreement.

Recommendations, care and warnings

The use of a floor polisher in the area next to the chair should be avoided as this may lead to the sprinkling of cleaning liquids and oxidizing spots. Shock may also occur and damage the painted parts of the chair.

No accumulation of cleaning products should be allowed on any parts of the equipment.

PVC film should be used on the entire upholstery as leather clothes or fabric that let out dyes can cause stains on it. The headrest and the cervical support also require such protection, since chemical products used in patients' hair reduce the durability of these covering materials.

Fuse Replacement

Fuses must be replaced by an Authorized Dabi Atlante Technical Assistant.

Procedure in case equipment functioning is altered

In case the equipment presents any faults, firstly check whether they are listed under "Troubleshooting". If the problem cannot be solved, turn off the equipment, unplug it from the power source outlet and call for Dabi Atlante Authorized Technical Assistance.

Sensitivity to predictable environmental conditions in normal working situations

This equipment is not sensitive to magnetic, electric, electrostatic or pressure interferences as long as the

cleaning, maintenance, transportation and operation recommendations found in this manual are observed.

Procedures in case of equipment disposal

When the equipment is put out of use, it must be disposed of at an appropriate site (according to local regulations), thus preventing its inadequate use and environmental contamination.

Transportation and storage conditions

The equipment must be transported and stored:

- Carefully in order to prevent falls and impact.
- By keeping the side indicated by the arrow facing upwards.
- By observing maximum piling limits as indicated on the packaging.
- Protected against moisture, rain, water sprinkling and wet ground.

Transportation /storage environmental conditions	
Transportation and storage environmental temperature range	0°C to + 55°C
Transportation and storage relative humidity range	0% to 90% (non-condensing)
Atmospheric pressure range	500hPa to 1060 hPa (375 mmHg to 795 mmHg)

After installation

Keep the equipment protected from sunlight and rain.

Environmental operation conditions	
Environmental temperature functioning range	+10°C to +35°C
Environmental temperature range recommended by Dabi Atlante	+21°C to + 26°C
Functioning relative humidity range	30% to 75% (non-condensing)
Atmospheric pressure range	700 hPa to 1060 hPa (525 mmHg to 795 mmHg)

Environmental accommodation operation conditions (between operations)

Environmental temperature accommodation range	+5°C to +45°C
Environmental temperature range recommended by Dabi Atlante	+15°C to +30°C
Functioning relative humidity range	30% to 75% (non-condensing)
Atmospheric pressure range	700 hPa to 1060 hPa (525 mmHg to 795 mmHg)

Preparation prior to delivery

This equipment has been revised and approved according to the Dabi Atlante Quality Inspection and Control Program for better performance.

Dabi Atlante Authorized Service Network

Installation and all maintenance service for Dabi Atlante equipment must be given by Dabi Atlante Authorized Technical Assistance, otherwise warranty will be compromised. Use the attached insert or access the website www.dabi.com.br to find the nearest Dabi Atlante Authorized Technical Assistant.

Symbols

Use the icons below to identify the symbols on your equipment:

	Indicates the terminals for grounding protection.
	Attention: warns that accompanying documents must be consulted.
	Type-B equipment
	"Fragile" – Located on the packaging side. It determines that equipment must be carefully transported in order to prevent falls or shocks.
	"Protect against humidity" – Located on the packaging side. It determines that the equipment must be protected against any type of humidity during transportation and storage.
	"This side up" – Located on the packaging side. It determines that the packaging must be handled by keeping the side shown by the arrow facing upwards.
	"Maximum piling" – Located on the packaging side. It determines the maximum number of boxes that can be piled up during transportation and storage.
	"Temperature limit" – It determines the temperature limits at which the equipment must be stored or transported.

Final recommendations

In order to ensure greater durability for your equipment, use only original Dabi Atlante replacement parts, which are in accordance with the technical standards and specifications required by the manufacturer. Also, use only the services provided by the Dabi Atlante Authorized Technical Assistance Network as it counts on trained assistants and specific tools for the appropriate maintenance of your equipment.

Classification and type

- According to Standard IEC 60601-1: Class I, Type B – Intermittent operation.
- According to Directive 93/42: Class I.

Equipment Warranty

The "Warranty Terms" are found on the back of the certificate accompanying the product. Any alterations or concessions involving the warranty as well as pertaining conditions and authorizations, either oral or written, are definitely not permitted if not previously documented and approved of by Dabi Atlante.

Optional, serial and non-applicable items

Items	Versa		Croma		Galla	
	Hydraulic System	Spindle System	Hydraulic System	Spindle System	Hydraulic System	Spindle System
Incorporated foot control	S	S	S	S	S	S
Electronic movements	S	S	S	S	S	S
Bi-articulated headrest	S	S	S	S	S	S
Retractable right arm	S	S	-	-	-	-
Right arm	-	-	O	O	O	O
Light activation by the foot control	O	O	O	S	S	S
Zero position	O	O	O	S	S	S
Programmable work position	-	-	O	S	S	S
Three programmable work positions	-	-	-	S	-	S
Sound indication definition	-	-	-	S	-	S
Soft start	-	-	-	S	-	S
Light automatic shut off when zero position is activated	-	-	-	S	-	S
Digital system (ZP + Light Activation + 3 Programmable WPs + SP + Cuspidor Activation + Digital Display)	-	-	-	-	O	-

S = serial item

O = optional item

- = not applicable

Technical Features

	Versa		Croma		Galla		Galla (Digital)
	Hydraulic System	Spindle System	Hydraulic System	Spindle System	Hydraulic System	Spindle System	Hydraulic System
Mains voltage	127/220/240 VAC +/- 10%	110/127/220/240 VAC +/- 10%	110/127/220/240 VAC +/- 10%	110/127/220/240 VAC +/- 10%	110/127/220/240 VAC +/- 10%	110/127/220/240 VAC +/- 10%	110/127/220/230 240 VAC +/- 10%
Frequency	50/60 Hz	50/60 Hz	50/60 Hz	50/60 Hz	50/60 Hz	50/60 Hz	50/60 Hz
Powering Fuse	6A (110/127V) 3A (220/240V)	10A (110/127V) 5A (220/240V)	6A (110/127V) 3A (220/240V)	10A (110/127V) 5A (220/240V)	6A (110/127V) 3A (220/240V)	10A (110/127V) 5A (220/240V)	6A (110/127V) 3A (220/240V)
Electronic circuit fuse	-	10A	-	10A	-	10A	-
Light fuse	-	6A	-	6A	-	6A	6A
Consumed power	380 VA	237 VA	780 VA	237 VA	780 VA	237 VA	780 VA
Electronic circuit	-	12 VDC	-	12 VDC	24 VDC	12 VDC	9/17/24 VDC
Length	1800mm	1800mm	1870mm	1870mm	1870mm	1870mm	1870mm
Width	580mm	600mm	600mm	600mm	660mm	660mm	660mm
Seat height	min. 340 max. 730	min. 340 max. 730	min. 390 max. 710	min. 390 max. 710	min. 385 max. 835	min. 440 max. 740	min. 385 max. 835
Backrest height	min. 1040 max. 1430	min. 1040 max. 1430	min. 1040 max. 1430	min. 1040 max. 1430	min. 1040 max. 1430	min. 1040 max. 1430	min. 1040 max. 1430
Backrest inclination	0° a 80°	0° a 80°	0° a 80°	0° a 80°	5° a 80°	0° a 80°	5° a 80°

Note: Fuse/spindle system corresponds to the elevation system of the chosen chair.

Technical responsibility by: Leonel Issa Halak - CREA-SP 0600256918

Anvisa registration n°: 10101130057

Troubleshooting

Faults	Causes	Solutions
Chair is completely inoperative	No power supply	Await normalization
	Master switch is off	Turn it on
	Fuse is burned out	Replace it
	Circuit breaker is off	Turn it on

Representante legal CE:

MPC International S.A - 26, Boulevard Royal - L-2449 - Luxembourg - Fone: +49 431-7028 219 - Fax: +49 431-7028 202

www.dabi.com.br
Tel.: +55 (16) 3512 1212