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Owner's Manual

Reflex

Português.....03

English.....07

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Contents

Legend	07
Reflex Light	07
Installation	07
Additional procedures prior to equipment use/ reutilization	07
Operation	07
Cleaning	07
Maintenance	07
Preventive maintenance	07
Corrective maintenance	08
Circuit diagrams, list of parts, components, others	08
Recommendations and care	08
Removal of scratches on the transparent shield	08
Bulb replacement	08
Bulb durability	08
Fuse replacement	08
Procedures in case equipment functioning is altered	08
Sensitivity to predictable environmental conditions in normal working situations	08
Procedures in case of equipment disposal	08
Transportation and storage conditions	08
After installation	08
Preparation prior to delivery	09
Dabi Atlante Authorized Service Network	09
Symbols	09
Final recommendations	09
Classification and type	09
Equipment warranty	09
Technical features	09
Troubleshooting	10

Presentation

Dear Customer,

Welcome!

We are proud of your preference for our products. Your familiarization with the **Reflex** light will help maximize your equipment's performance and increase its durability and benefits. Therefore, before starting your equipment, be sure to read this Manual very attentively.

Important

All the information, illustrations and specifications in this manual are based on data existing at the time of its publication. Rights are reserved to make alterations at any time, both to the product and to this Manual, with out a previous notice.

Legend

Reflex Reflector (fig. A)

- ① Transparent Shield
- ② Bulb Shield
- ③ Mirror
- ④ Handle
- ⑤ Flex Arm Assembly
- ⑥ Rigid Arm Assembly
- ⑦ Bulb

Installation

This equipment must be unpacked and installed by an authorized Dabi Atlante technician, otherwise, warranty will be voided.

Additional procedures prior to equipment use/reutilization

Prior to equipment use/reutilization, follow the operation and cleaning instructions in this Manual.

Operation

Place the light at approximately 70cm from the operating field (fig. B) using the movements of the flex arm (fig. H).

Activate the light using the foot control

Turn on the main switch on the chair base. Press the light activation key on the chair's foot control (item 1, fig. C) to turn the light on/off.

The light can also be activated through some of the Dabi Atlante delivery unit and cuspidor models. Refer to such products' manuals for further information.

Cleaning

Clean the equipment surface with a clean soft cloth moistened in Aplic Odonto by Dabi Atlante or another product with similar chemical properties.

The use of other chemical products is not recommended as it may damage the equipment.

Maintenance

Preventive maintenance

In order to reduce equipment faults and increase durability, call Dabi Atlante Authorized Technical Assistance and set a regular preventive maintenance plan.

Corrective maintenance

In case the equipment presents any faults that are not listed in "Troubleshooting", contact Dabi Atlante Authorized Technical Assistance.

⚠ Do not open the equipment and/or try to repair it as such action may aggravate the problem or even generate further malfunctioning.

Circuit diagrams, list of parts, components, others

In case they are necessary, ask Dabi Atlante for electric diagrams and/or lists of parts, components or other information, which will be supplied by agreement.

Recommendations and care

Removal of scratches on the transparent shield

Use automobile paint polisher to remove or minimize slight scratches and abrasion marks.

Bulb replacement

Turn off the main switch on the chair base.
 Make sure that the light has cooled off.
 Using a 1/8x2" Phillips screwdriver, remove the frontal cover (Fig. D).
 Also using a screwdriver, loosen the socket kit screw, noting the positioning of the plate attached to the light bulb (Fig. E).
 Remove the burned out bulb (Fig. F).
 Disconnect the bulb wire from the terminal (Fig. G).
 Place the new bulb and follow the contrary sequency to that used for disassembly, paying attention to the positioning of the plate attached to the bulb.

⚠ Do not touch the light bulb. In case contact occurs, clean it with alcohol. Check if the bulb filament is in the vertical position in the same direction of the glass facets.

Bulb durability

Bulb durability is of approximately 400 hours, and 3% of them may prematurely burn out during this period.

Fuse replacement

Fuse replacement must be made by an Authorized Dabi Atlante Technical Assistant.

Procedures in case equipment functioning is altered

In case the equipment presents any faults, firstly check whether they are listed under "Troubleshooting".

In case the problem cannot be solved, turn off the equipment, unplug it from the power source outlet and call Dabi Atlante Authorized Technical Assistance.

Sensitivity to predictable environmental conditions in normal working situations

This equipment is not sensitive to magnetic, electric, electrostatic or pressure interferences as long as the cleaning, maintenance, transportation and operation recommendations found in this manual are observed.

Procedures in case of equipment disposal

When the equipment is put out of use, it must be disposed of at an appropriate site (according to local regulations), thus preventing its inadequate use and environmental contamination.

Transportation and storage conditions

- The equipment must be transported and stored:
- Carefully in order to prevent falls and impact.
 - By keeping the side indicated by the arrow facing upwards.
 - By observing maximum piling limits as indicated on the packaging.
 - Protected against moisture, rain, waters sprinkling and wet ground.

Transportation/storage environmental conditions	
Transportation and storage environmental temperature range	0°C to + 55°C
Transportation and storage relative humidity range	0% to 90% (non-condensing)
Atmospheric pressure range	500hPa to 1060 hPa (375 mmHg to 795 mmHg)

After installation

Keep the equipment protected from sunlight and rain.

Environmental operation conditions	
Environmental temperature functioning range	+10°C to +35°C
Environmental temperature range recommended by Dabi Atlante	+21°C to + 26°C
Functioning relative humidity range	30% to 75% (non-condensing)
Atmospheric pressure range	700 hPa to 1060 hPa (525 mmHg to 795 mmHg)

Environmental accommodation operation conditions (between operations)	
Environmental temperature accommodation range	+5°C to +45°C
Environmental temperature range recommended by Dabi Atlante	+15°C to +30°C
Functioning relative humidity range	30% to 75% (non-condensing)
Atmospheric pressure range	700 hPa to 1060 hPa (525 mmHg to 795 mmHg)

Preparation prior to delivery

This equipment has been revised and approved according to the Dabi Atlante Quality Inspection and Control Program for better performance.

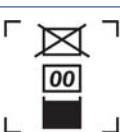
Dabi Atlante Authorized Service Network

Installation and all maintenance service for Dabi Atlante equipment must be given by Dabi Atlante Authorized Technical Assistance, otherwise warranty will be compromised.

Use the attached insert or access the web site www.dabi.com.br to find the nearest Dabi Atlante Authorized Technical Assistant.

Symbols

Use the icons below to identify the symbols on your equipment:

	"Fragile" Located on the packaging side. It determines that equipment must be carefully transported in order to prevent falls or shocks.
	"Protect against humidity" Located on the packaging side. It determines that the equipment must be protected against any type of humidity during transportation and storage.
	"This side up" Located on the packaging side. It determines that the packaging must be handled by keeping the side shown by the arrow facing upwards.
	"Maximum piling" Located on the packaging side. It determines the maximum number of boxes that can be piled up during transportation and storage.

	"Temperature limit" It determines the temperature limits at which the equipment must be stored or transported.
--	--

Final recommendations

In order to ensure greater durability for your equipment, use only original Dabi Atlante replacement parts, which are in accordance with the technical standards and specifications required by the manufacturer.

Also, use only the services provided by the Dabi Atlante Authorized Technical Assistance Network as it counts on trained assistants and specific tools for the appropriate maintenance of your equipment.

Classification and type

- According to Standard IEC 60601-1: Class I, Type B – Intermittent operation.
- According to Directive 93/42: Class I.

Equipment Warranty

The "Warranty Terms" are found on the back of the certificate accompanying the product. Any alterations or concessions involving the warranty as well as pertaining conditions and authorizations, either oral or written, are definitely not permitted if not previously documented and approved of by Dabi Atlante.

Technical features

Specifications	Reflex
Illumination field	Approximately 8x16 cm at 70cm of distance
Halogen bulb	H3 12 V.A.C. 55W
Installation options	Chair, delivery unit and bench
Voltage	12 V.A.C. powered by the chair
Frequency	50/60HZ
Fuse	6A - 5x20 - Located on the chair
Power consumption	75VA
Light intensity	6000 to 22000 LUX

Troubleshooting

Fault	Cause	Solutions
Ligth will not go on	No power supply	Await supply normalization
	Chair's main switch is off	Turn it on
	Burned out bulb	Replace it

Technical responsibility by: Leonel Issa Halak – CREA-SP 0600256918

CE legal representative:

MPC International S.A - 26, Boulevard Royal - L-2449 - Luxembourg - Phone: +49 431-7028 219 - Fax: +49 431-7028 202

www.dabi.com.br
Tel.: (16) 3512 1212